
EN AirLife™ Edith HMEs, disposable Instructions for Use

REF	557056200	AirLife™ Edith 500 HME, Disposable	557056200	AirLife™ Edith 1000 HME, Disposable	557057200	AirLife™ Edith 1500 HME, Disposable	557058000	AirLife™ Edith 1500 HME with Membrane, Disposable	557044500	AirLife™ EdithFlex HME with Integrated Flexible Tube, Disposable
REF	Catalogue number		Caution		Batch code		Consult instructions for use		Quantity	

LOT	Batch code		Consult instructions for use		Expiry date		Single use	
QTY	Quantity		Expiry date		Single use		Single use	
Manufacturer	Manufacturer		Single use		Single use		Single use	
U.S. Federal law restricts this device to sale by or on the order of a physician.	U.S. Federal law restricts this device to sale by or on the order of a physician.		Single use		Single use		Single use	

Material
Housing: Polypropylene (PP).
HME element: Plastic
Not made with natural rubber latex.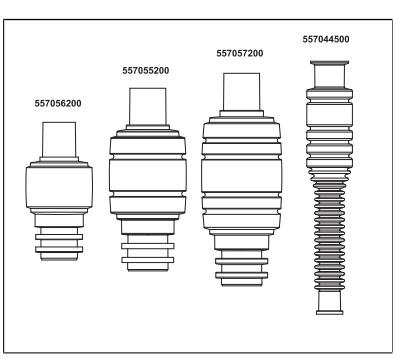

HME= Heat and moisture exchanger

Intended use
Edith HMEs are single use Heat and Moisture Exchanger that transfer heat and humidity from the gas expired by the patient to the inspired gases. The Edith HMEs shall be used to humidify patients that are artificially ventilated during intensive care or anesthesia.

Instructions for use
Read these instructions carefully before using the product. Methods of connection: Edith HMEs shall be placed between the proximal end of the artificial airway and the Y-piece of the breathing circuit. Edith HMEs shall be changed every 24 hours or more frequently as required. Shelf life: Max. 3 years. Expiration date printed on package.

Contraindications
Edith HMEs are contraindicated in patients producing fulminating, frothy secretions within their airways and lungs. Edith HMEs shall not be used on patients with very small tidal volumes, for example neonates or very small children. Edith HMEs shall not be used together with active humidifiers or nebulizers.

Precautions
All tubing and connections to Edith HMEs shall be properly fixed and checked for leakage prior to use. Before use, verify that the HME has no occlusions and that air will flow through it. Compensation of ventilation may be necessary when using Edith HMEs since dead space will be added to the system. During the use of Edith HMEs, the patient shall be closely monitored and proper airway care administered if complications arise. The Edith HMEs must be changed between patients.


Warnings
Single-use products are not designed to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy and/or system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization and/or reuse.

Technical data

	557056200	557055200	557057200 557058000**	557044500
Minimum VT (ml)	70	110	150	150
Maximum VT (ml)	500	1000	1500	1500
Dead space (ml)	17	28	38	90 ¹⁾
Weight (g)	6	8	9	20
Moisture to patient (mg H ₂ O/l)	30	30	30	30
Moisture loss (mg H ₂ O/l)	7.6	7.6	7.6	7.6
Pressure drop measured at V _T (mm Hg)	250	500	1000	1000
Pressure drop, kPa (cm H ₂ O)				
- at 30 l/min	0.08 (0.8)	0.1 (1.0)	0.1 (0.5)	0.05 (0.5)
- at 60 l/min	0.2 (2.0)	0.25 (2.5)	0.25 (2.5)	0.14 (1.4)

¹⁾ The integrated flexible tube included. ²⁾ See additional Information Sheet for Dead Space.

Further information
For further information, please contact your local sales representative or visit www.mylife.com.

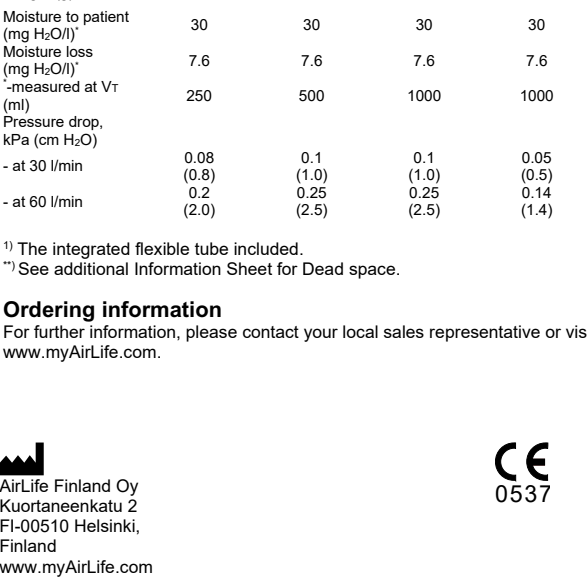Australian Sponsor
AirLife Australia Holdings Pty Ltd.
PO BOX 57
NORTH RYDE NSW 657
Australia

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