Medisorb™ EF Multi-Absorber, Disposable

Instructions for Use

REF M1173311 Misorb™ EF Multi-Absorber, Disposable

REF	Catalogue number		Consult instructions for use
LOT	Lot number		Follow instructions for use
QTY	Quantity		Keep dry
	Manufacturer		Fragile, handle with care
	Date of manufacture		This side up
	Expiry date		Keep away from sunlight
	Temperature limit		Warning
R_X ONLY	U.S. Federal law restricts this device to sale by or on the order of a physician.		

Intended use

The Misorb™ EF Multi-Absorber is a disposable product intended for use with the GE Healthcare Advanced Breathing System (later ABS), the GE Healthcare EZchange manifold, the GE Healthcare Compact Block, and the GE Healthcare Compact Block II (later Compact Block).

The Multi-Absorber should only be used with air, oxygen, nitrous oxide, halothane, enflurane, isoflurane, desflurane and sevoflurane.

The device is intended to be used under constant attention of qualified professional healthcare personnel.

Technical description

Weight

Total weight: 890 g
Absorbent weight: 720 g

Volume

Canister: 1250 ml
Max. absorbent: 960 ml

Absorbent

Granule size: 2.5–5 mm
Moisture content: 12–19%
pH: <12.5
Sodium hydroxide (NaOH, 215-185-5): <1%
Calcium hydroxide (Ca(OH)₂, 215-137-3): >75%

Materials

Casing: PP (Polypropylene)
Dust Filter: PUR (Polyurethane)
Resistance (at 60 l/min)
Mounted on ABS (insp./exp.):
0.37 kPa/0.47 kPa
Mounted on Compact Block (insp./exp.):
0.38 kPa/ 0.34 kPa
Mounted on EZchange (insp./exp.):
0.41 kPa/ 0.48 kPa

Instructions for use

1. Read these instructions carefully before using the product.
2. Observe that when handling and rotating the Multi-Absorber canister channeling of the absorbent might occur. The channeling can reduce the CO₂ absorption capacity of the product resulting in early CO₂ breakthrough. Channeling can be removed by tapping the canister gently on a hand or on a table 3–4 times in the orientation in which it will be installed (see Figure 1).
3. Before using the multi-absorber, remove the bottom seal.

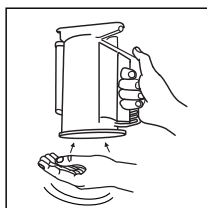
Figure 1

4. Make sure the connecting surfaces are free of dust, the Multi-Absorber is properly installed and the breathing circuit is checked for leakage and occlusion prior to use.

ABS

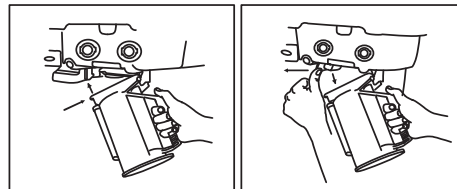
Figure 2

Figure 3

Connecting to ABS (see Figure 2)

1. Turn the Multi-Absorber so that the legs are on top.
2. Hook the Multi-Absorber legs over the support pins on the ABS.
3. Rotate the Multi-Absorber into a vertical position until it locks into place.

Warning

It is contraindicated to remove the Multi-Absorber from the ABS during an anesthetic procedure unless the ABS has the optional EZchange manifold installed.

Removing from ABS (see Figure 3)

1. Hold the Multi-Absorber by the handle and push on the release latch to unlock the absorber.
2. Remove the absorber by tilting it downward and off the two support pins.
3. To avoid water spillage, keep the Multi-Absorber legs up until the water trap column in the Multi-Absorber is emptied.

EZchange

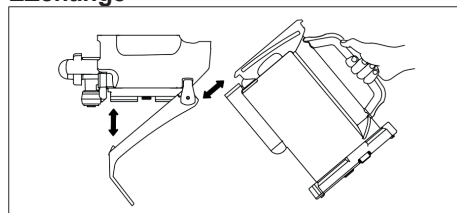
Figure 4

Connecting to EZchange manifold (see Figure 4)

1. Release the absorber cradle and allow it to rotate down.
2. Slide the Multi-Absorber into the cradle.
3. Grasp the Multi-Absorber handle and rotate the absorber into a vertical position until it locks into place.

Replacing Multi-Absorber during ventilation

The Multi-Absorber is connected to the EZchange manifold through a valve system.

The valve system makes it possible to replace the Multi-Absorber during ventilation without disturbing the gas flow to the patient.

Warning

Removal of the Multi-Absorber while the anesthesia system is delivering gas to a patient allows carbon dioxide to build up in the breathing system.

Removing from EZchange manifold (see Figure 4)

1. Release the absorber cradle and allow it to rotate down with the absorber.
2. Remove the absorber by sliding it towards the operator until it clears the cradle.
3. To avoid water spillage, keep the Multi-Absorber legs up until the water trap column in the Multi-Absorber is emptied.

Compact Block

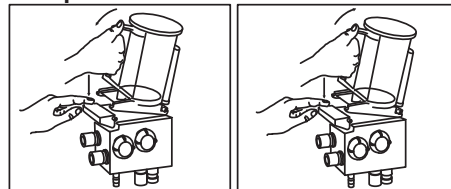
Figure 5

Figure 6

Connecting to Compact Block (See Figure 5)

1. Hook the Multi-Absorber legs opposite the handle under the metal pins on the Compact Block.
2. Push the release lever down.
3. Push the Multi-Absorber down until it locks into place.

Replacing Multi-Absorber during ventilation

The Multi-Absorber is connected to the Compact Block through a piston valve system.

The piston valve system makes it possible to replace the Multi-Absorber during ventilation without disturbing the gas flow to the patient.

Warning

Removal of the Multi-Absorber while the anesthesia system is delivering gas to a patient allows carbon dioxide to build up in the breathing system.

Dismounting from Compact Block (see Figure 6)

1. Grab the handle.
2. Push the release lever down.
3. Lift the Multi-Absorber upwards.

When to change Medisorb EF Multi-Absorber

The absorption of carbon dioxide is indicated by a gradual color change of the soda lime. Discard the product as soon as the color change has occurred. The intensity of the color change may vary from one procedure to another. The color change of the soda lime is only a rough indicator. Use carbon dioxide monitoring to determine when to change the Medisorb EF Multi-Absorber. The color change is reversible and the exhausted material will change back to its original color. Saturated Medisorb EF Multi-Absorber should therefore be disposed of immediately after use.

A typical carbon dioxide absorption capacity of the Medisorb EF Multi-Absorber is 6 hours for inspired CO₂ to reach 0.5% with a minute ventilation of 5 l/min, (a respiratory rate of 10 breaths/min and a tidal volume of 500 ml) a fresh gas flow of 500 ml/min and a carbon dioxide output from the patient of 190 ml/min. Hours of use depend on fresh gas flow and patient carbon dioxide output.

Operating room personnel must determine the remaining absorptive capacity of the Medisorb EF Multi-Absorber immediately at the end of a surgical procedure. Excessive use of oxygen flush induces dehydration of the Medisorb EF Multi-Absorber and inhibits its ability to absorb carbon dioxide and may cause a premature color change.

There may be some amount of absorbent in the bottom of the absorber that does not change color; this is a natural occurrence.

It is recommended to use water traps in the patient circuit during low flow anesthesia (fresh gas flow less than 1 l/min) and when excessive condensed water accumulates in the limbs. Use water traps in both expiratory and inspiratory limbs to remove water condensation.

Maintenance

Disposal

Saturated Medisorb EF Multi-Absorber should be disposed of immediately after use. Keep all Medisorb EF Multi-Absorbers that have been used with highly flammable anesthetics away from heat, sparks and open flames, as residual amounts of these materials will be present and may cause a fire hazard. Disposal of a used Medisorb EF Multi-Absorber must be determined by the end user in compliance with federal state, provincial and local regulations. Specific disposal methods vary with the chemical (e.g., anesthetic agent, acid-base vapors, etc.) or biological agents used with the absorbent by the end user.

Storage

The unopened Medisorb EF Multi-Absorber package must remain unopened and be stored in a dry and clean environment at a temperature from 0°C/+32°F to +35°C/+95°F. Avoid direct sunlight (UV light). Avoid freezing which may reduce CO₂ absorption performance. Once the Medisorb EF Multi-Absorber package is opened, its contents should be used immediately as the performance will deteriorate if storage is continued. The package must be protected from physical damage and water. Under these conditions, the Medisorb EF Multi-Absorber will retain its carbon dioxide absorption capacity for two years. The expiration date of the Medisorb EF Multi-Absorber is printed on the product.

Warnings

- Hazard and Precautionary statements of the soda lime: H315, Causes skin irritation.
- P264, Wash hands and face thoroughly after handling.
- P280, Wear protective gloves/protective clothing/eye protection/face protection.
- P302/P352, IF ON SKIN: Wash with plenty of soap and water.
- P332/P313, If skin irritation occurs: Get medical advice/attention.
- P362/P342, Take off contaminated clothing and wash it before reuse.
- Constant attention by a qualified professional is needed whenever this product is used.
- The Medisorb EF Multi-Absorber is a sealed unit, and should not be opened for any reason. No modification is allowed.
- Avoid contact of contents of Medisorb EF Multi-Absorber with eyes, skin or clothing. Avoid creating and breathing airborne dust that may cause irritation. In the event of skin or eye contact, immediately rinse the affected area with water and seek medical assistance.
- The color change of the soda lime is only a rough indicator. Use carbon dioxide monitoring to determine when to change the Medisorb EF Multi-Absorber. Color change is reversible and the exhausted material will change back to its original color. Saturated Medisorb EF Multi-Absorber should therefore be disposed of immediately after use.
- Excessive use of oxygen flush induces dehydration of the Medisorb EF Multi-Absorber and inhibits its ability to absorb carbon dioxide.
- If a Medisorb EF Multi-Absorber is left mounted overnight, make sure that all gas flows are stopped. Otherwise the absorber may dry out and lose its

activity. If the Medisorb EF Multi-Absorber dries out completely, it may give off CO (carbon monoxide) when exposed to anesthetic agents. Use carbon dioxide monitoring to determine when to change the Medisorb EF Multi-Absorber.

- For safety, always replace the absorber when its state of desiccation is unknown.
- The Medisorb EF Multi-Absorber must not be used with chloroform or trichloroethylene. The Medisorb EF Multi-Absorber may react with chloroform/trichloroethylene, producing sodium formate/dichloroethylene, carbon monoxide and/or phosgene, which are harmful for the patient.
- Inhaled anesthetic agents in contact with strong bases are reported to be able to degrade to produce breakdown products of unknown toxicity in humans. For further information, contact the manufacturer of the inhaled anesthetic agent in question.
- The Medisorb EF Multi-Absorber is not an antimicrobiological filter or inhibitor. Based on this information, it is the responsibility of the clinician to take appropriate measures regarding patient cross contamination when required.
- Vital Signs™ recommends that the Vital Signs filter is used in order to minimize the cross contamination and the risk of foreign particles reaching the patient.
- When the disposable Multi-Absorber is used on an ABS that has an attached EZchange Module with condenser and the breathing circuit flow rate is greater than 60 l/min, the total breathing circuit resistance to flow may exceed 0.6 kPa. Verify proper operation before use.
- It is recommended to use water traps in the patient circuit during low flow anesthesia (fresh gas flow less than 1 l/min) and when excessive condensed water accumulates in the limbs. Use water traps in both expiratory and inspiratory limbs to remove water condensation.
- Use only Vital Signs validated supplies and accessories approved by AirLife for proper performance of the anesthesia system. Using products other than those approved may result in improper performance of the anesthesia system.
- Keep all Medisorb EF Multi-Absorbers that have been used with highly flammable anesthetics away from heat, sparks and open flames, as residual amounts of these materials will be present and may cause fire hazard.

Cautions

For information about the soda lime, refer to the appropriate Safety Data Sheet.

Local authority regulations must be investigated before disposal of the product.

Ordering information

For further information, please contact your local sales representative or visit myAirLife.com.

AirLife Finland Oy
Kuortaneenkatu 2
FI-00510 Helsinki, Finland
www.myAirLife.com

0537

Made in UK

Australian Sponsor
AirLife Australia Holdings Pty Ltd.
PO BOX 97
NORTH RYDE BC NSW 1670
Australia

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