[EN]


Vital Signs™ Gas sampling line, disposable

[REF]	73306	Vital Signs™ Gas Sampling Line, Disposable, 6m/20ft
[REF]	Catalogue number	 Caution
[LOT]	Lot number	 Consult instructions for use
[QTY]	Quantity	 Follow instructions for use
	Manufacturer	 Single use
	Date of manufacture	 U.S. Federal law restricts this device to sale by or on the order of a physician.

Material
TPE Thermoplastic elastomer

Intended use

These gas sampling lines are used with GE Healthcare Compact Airway modules and CARESCAPE® Respiratory modules for monitoring CO₂, O₂, N₂O and anesthetic agents and with E-miniC for monitoring CO₂. Use when monitoring of respiratory gases during MRI (Magnetic Resonance Imaging) or in other application, when long sampling line is required. These devices are indicated for use by qualified medical personnel only.

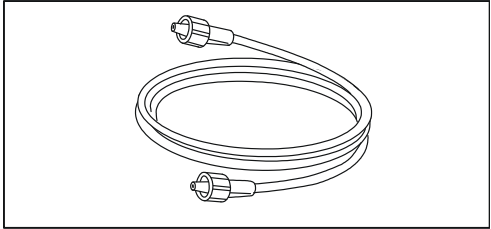Instructions for use

- Connect the sampling line to the monitor and the appropriate gas sampling port. Securely attach all connections by using a twisting action to ensure a leak free fit.
- Check all connections for leaks.
- Do not reuse.

Warnings

- Single-use products are not designed to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy and/or system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization and/or reuse.
- No modification is allowed.

Disposal

This product and package should be disposed of according to local environmental and waste disposal regulations.

Technical description

For technical description, see the host device user guide or manual, or contact your local sales representative.

Ordering information

For further information, please contact your local sales representative or visit www.myAirLife.com.

	
AirLife Finland Oy Kuortaneenkatu 2 FI-00510 Helsinki, Finland www.myAirLife.com	
Made in China	
Australian Sponsor AirLife Australia Holdings Pty Ltd. PO BOX 97 NORTH RYDE BC NSW 1670 Australia	
1000007931 Rev.A 2025/02	

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