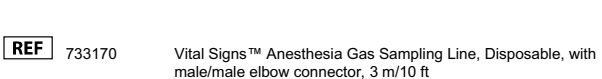
[EN]


Vital Signs™ Anesthesia gas sampling line, disposable with elbow Luer connector

Instructions for Use

[REF]	733170	
[REF]	Catalogue number	
[LOT]	Lot number	
[QTY]	Quantity	
	Manufacturer	
	Date of manufacture	

Intended use

This gas sampling line can be used for monitoring CO₂, O₂, N₂O and anesthetic agents. Use with D-lite™ or Pedi-lite sensors. Products are in indirect patient contact via gas passing through sampling site. These products are indicated for use by qualified medical personnel only.

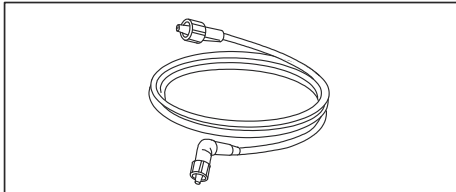Material

PE / PVC

Instructions for use

1. Connect the sampling line to the monitor and the elbow connector to the gas sampling port on the D-lite™ flow sensor.
2. Check all connections for leaks.

Warnings

- Use only the gas sampling port of the D-lite™, if using Patient Spirometry.
- No modification is allowed.
- Single-use products are not designed to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy and/or system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization and/or reuse.

Disposal

This product and package should be disposed of according to local environmental and waste disposal regulations.

Technical description

For technical description see the host device user guide or manual, or contact your local sales representative.

Ordering information

For further information, please contact your local sales representative or visit www.myAirLife.com.

[FR]	
	
Made in Estonia	
Australian Sponsor AirLife Australia Holdings Pty Ltd. PO BOX 97 NORTH RYDE BC NSW 1670 Australia	
1000007932 Rev.A 2025/02	

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