
EN	Vital Signs™ Exhaust line with Colder fitting, disposable
REF	8004482 Vital Signs™ Exhaust Line with Colder Fitting, 1 m/41 in 8004483 Vital Signs™ Exhaust Line with Colder Fitting, 0.18 m/7 in
REF	Catalogue number
LOT	Lot number
QTY	Quantity
	Manufacturer
	The CE marking declares compliance with the applicable EU legislation.
	Consult instructions for use
	Follow instructions for use
	Single use
	Gas exhaust
	U.S. Federal law restricts this device to sale by or on the order of a physician.
	Indicates the authorized representative in Switzerland

Intended use

Exhaust line for the gas return or scavenging. Use the short line (1) for gas return in the GE Healthcare anesthesia machine. Use the long line (2) for monitors having the sample gas out port on the rear panel. Products are in indirect patient contact via gas passing through sampling site. These products are indicated for use by qualified medical personnel only.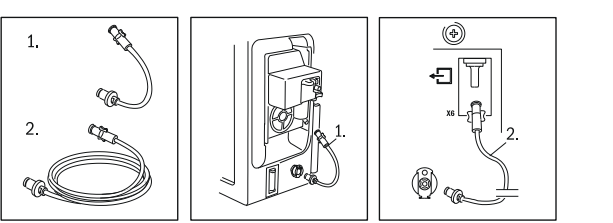

Material

PVC

Instructions for use

1. Connect the female Luer end into the sample gas out port in the module and the straight Colder male end to the female Colder connector in the anesthesia machine. 2. Connect the female Luer end into the sample gas out port on the rear panel of the monitor and the straight Colder male end to the female Colder connector in the anesthesia machine.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myairlife.com and the competent authority of the Member State in which the user and/or patient is established.

Maintenance

Do not wash or disinfect exhaust lines.

Warning

- No modification is allowed. - Single-use products are not designed to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy and/or system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization and/or reuse. - This disposable product must always be replaced if it becomes soiled or if there is any visual sign of damage.

Disposal

This product and package should be disposed of according to local environmental and waste disposal regulations.

Technical description

For technical description see the host device user guide or manual or contact your local sales representative.

Ordering information

For further information, please contact your local sales representative or visit myAirLife.com.

	0537
AirLife Finland Oy Kuortaneentie 2, FI-00510 Helsinki, Finland www.myAirLife.com	
Made in Estonia	

Australian Sponsor: North Australia Holdings Pty Ltd. PO BOX 97 NORTH RYDE BC NSW 1670 Australia	
MediEngvo Switzerland Gothardstrasse 28 6302 Zug Switzerland	

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