[EN]

Vital Signs™ Gas Sampling Line, Coextruded, Disposable

Instructions for Use


[REF] GSLCO31010 Vital Signs™ Gas Sampling Line, Coextruded, 3 m/10 ft, 10' pkg, Disposable

[REF]	Catalogue number		Caution
[LOT]	Lot number		Consult instructions for use
[QTY]	Quantity		Follow instructions for use
	Manufacturer		Single use
	Date of manufacture		U.S. Federal law restricts this device to sale by or on the order of a physician.

Indications for Use

The gas sampling lines are intended to connect from a port in the breathing circuit to the expired gas monitor. These gas sampling lines are used with GE Healthcare Compact Airway modules and CARESCAPE™ Respiratory modules for monitoring CO₂, O₂, N₂O and anesthetic agents and with E-minic® for monitoring CO₂.

These devices are indicated for use by qualified medical personnel only.

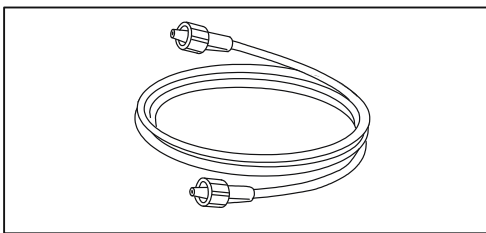Material

Coextruded: PE/PVC Polyethylene/Polyvinyl chloride

Instructions for use

1. Connect the sampling line to the monitor and the appropriate gas sampling port. Securely attach all connections by using a twisting action to ensure a leak free fit.
3. Do not reuse.

Warnings

- This product shall not be used if the integrity of the packaging has been compromised.
- This disposable product must always be replaced if it becomes soiled or if there is any visual sign of damage.
- Single-use products are not designed to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy and/or system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization and/or reuse.
- No modification is allowed.

Disposal

This product and package should be disposed of according to local environmental and waste disposal regulations.

Technical description

For technical description, see the host device user guide or manual, or contact your local sales representative.

Ordering information

For further information, please contact your local sales representative or visit www.myAirLife.com.

	
AirLife Finland Oy Kuortaneenkatu 2 FI-00510 Helsinki, Finland www.myAirLife.com	0537

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