Oxygen Mask Non-Rebreather

EN Oxygen Mask Non-Rebreather

DEVICE DESCRIPTION: The device is non-sterile, disposable and for single-patient multiple-use. A non-rebreather mask is a high concentration oxygen mask placed over a patient’s nose or mouth.

INTENDED PURPOSE: A non-rebreathing mask is a device fitting over a patient’s face to administer oxygen. It utilizes one-way valves to prevent the patient from rebreathing previously exhaled gases.

INDICATIONS FOR USE: For patient’s requiring supplemental oxygen. To treat hypoxia, relieve shortness of breath and pre-oxygenate for surgical procedures in spontaneously breathing patients.

ENVIRONMENT: Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities.

PATIENT TARGET GROUP: Pediatric, adult.

EXPECTED CLINICAL BENEFITS: The oxygen mask delivers up to 25 LPM of oxygen to reverse or prevent hypoxemia, and provide relief of shortness of breath in oral and nasal breathers.

CONTRAINDICATIONS: None Known.

WARNINGS:

- If oxygen is in use, do not use near flame or heat source.
- Position tubing to avoid strangulation.
- Device contains components that may present choking hazard.
- Patient may become hypoxic if oxygen flow is interrupted.
- Do not use flow rates less than 10 LPM.
- Keep excess tubing loosely coiled and out of the way to prevent kinking and tripping hazard.
- To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device before connecting.

CAUTIONS:

- Consult physician if patient develops infection, skin irritation or material sensitization.
- Do not place anything on supply tubing that may obstruct flow.

RESIDUAL RISKS: Refer to Warnings and Cautions.
NECESSARY USER QUALIFICATIONS: The end-user should receive training on the proper use of the device and be familiar with the indications for use, warningss and cautions as stated in the IFU.

Rx Only: Federal law (USA) reestricts this device to sale by or on the order of a licenser healthcare practitioner.

INSTRUCTIONS: Follow instructions to avoid risk of infection and contamination.

- Attach tubing to oxygen source and set flow to 10 LPM.
- Occlude valve between mask and reservoir bag until reservoir bag fills up.
- Place mask over nose and mouth.
- Adjust elastic strap and nose bar for a snug fit.
- Adjust the oxygen flow rate to ensure that the reservoir bag does not collapse during inspiration.
- Check for leaks at connection and that one-way valves work properly.

SAFE DISPOSAL: Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

INCIDENT REPORTING: Notice to the user that any serious incident that has occurred in relation to the device should be reported to the manufacturer, productquality@myairlife.com, and the competent authority of the Member State in which the user and/or patient is established.

PERFORMANCE CHARACTERISTICS:

- For use with Flow Rate 3 - 25 LPM
- For use with Bonded O2 Tubing adaptor

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