REF FC0010

en AirLife® Sterile Water for **INHALATION**, USP 1000 ml with Spikeable Cap and Hanger

Device Description: Sterile Water for Inhalation, USP is sterile, nonpyrogenic and contains no bacteriostatic or antimicrobial agents. Not made with natural rubber latex, polyvinyl chloride (PVC), or plasticizer diethylhexylphthalate (DEHP). External surfaces of bottle not sterile. Storage Temperature: 20°C to 25°C (68°F to 77°F). Avoid excessive heat. Protect from freezing. Discard unused portion. Sterile Water for Inhalation, USP contains: 1000 mL Sterile Water, pH range (5±0.5).

Indications for Use: The Sterile Water for Inhalation in 1000 mL Flexible Container is intended to provide a supply of sterile water to unfilled respiratory humidifier chambers. It is intended to be used in institutional and non-institutional care settings by qualified staff. The Sterile Water for Inhalation is intended for use in all patient populations including adults, pediatrics, infants, and neonates. The contents of these bottles are for use in respiratory therapy only.

⚠ Cautions:

- Squeeze and inspect bottle for any leaks.
- Discard if leaks are found.
- For inhalation therapy only, not for parenteral administration.
- Not for injection or irrigation.
- Do not use unless solution is clear and container is undamaged.
- Discard unused contents of open containers.
- Store at storage temperature until ready to use.
- For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.
- Open System ("Sistema Abierto") and Closed System ("Sistema Cerrado") graduations are for reference only.

Necessary User Qualifications: Federal law (USA) restricts this device to sale by or on the order of a physician or licensed practitioner. Dosage and administration as directed by physician.

Instruction for Use:

To Open (Use aseptic technique)

1. Unfold hanger.
2. Suspend the container from the eyelet support.
3. Holding the container at approximately a 45° angle, remove protective cap over rubber disc for spiking and swab with IPA.
4. For auto-feed transfer set: Insert spike well into the solution port.
5. For manual transfer set: Close control clamp of transfer set and insert spike well into the solution port.
6. Proceed according to respiratory transfer set and device manufacturer instructions.

How Supplied: Sterile Water for Inhalation, USP is supplied sterile, nonpyrogenic, in single dose LDPE (low density polyethylene) flexible containers as displayed in the table below.

Product Code	Product	Container Size (mL)
FC0010	Sterile Water for Inhalation, USP	1000

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