Vital Signs™

Head Positioner

Disposable Prone Position Head

Cushion with Mirror

Instructions for Use

**REF**

Catalogue number

**LOT**

Lot Number

**QTY**

Quantity

**LATEX**

Not made with natural rubber latex

**Single Use**

Single Use

**Exp. Date**

Exp. Date

**MD**

Medical Device

**EC REP**

Authorized Representative in the European Community

**Rx ONLY**

U.S. Federal law restricts this device to sales by or on the order of a physician

**Manufacturer**

Manufacturer

**Date of Manufacture**

Date of Manufacture

**Consult instructions for use**

Consult instructions for use

EN

English

Intended Use: A head positioner is a device intended to maintain the position of an anesthetized patient during prone position surgery.

Warnings:

- To prevent injury from possible shifting of the tracheal tube or other devices, assess patient for excessive contact pressure especially around patient's eyes, nose, and mouth.
- Do not rinse, soak, wash, or sterilize.

Preparation For Use

- Peel protective cover from mirror and discard. Place the mirror, in approximately its final position, with the reflective surface facing up.
- Intubate the patient in the supine position and place the Head Positioner on the face. Feed the tracheal tube through the Head Positioner mouth opening. Holding the Head Positioner firmly in place, turn the patient into the prone position. Reposition the mirror centering it under the cushion.

AU REP AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

 MedEnvoy
NL-IM-000000248
Prinses Margrietplantsoen 33
Suite 123
2595 AM, The Hague
The Netherlands

 SunMed, LLC
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

EC REP MT Promedt Consulting GmbH
Ernst-Heckel-Straße 7
66386 St. Ingbert
Germany

AirLife®

- Confirm the Head Positioner is properly fitted to the patient's face assessing for potential pressure points and assuring adequate clearance between the nose and the mirror. Confirm that turning the patient did not loosen circuit connections or cause the tracheal tube to kink or disconnect.
- To maintain neutral neck and head position, adjust the patient's body accordingly.
- Periodically reassess patient's position in the Head Positioner for pressure points, facial clearance and tracheal tube connection and patency.

This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.

A discontinued use of this product on the same patient, with periods of use separated by a period of non-use, is acceptable. The user must ensure that the product is not damaged or contaminated between usages on the same patient.

Notes:

- Discontinue use if allergic reaction is observed.
- This product is not sterile. This product should not be used/discontinued use if physical/functional deterioration, brittleness or discoloration occurs.
- For expiration date, reference product labeling.

Dispose per local and state environmental regulations.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myAirLife.com and the competent authority of the Member State in which the user and/or patient is established.

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