

BALLARD™ Q2 ORAL CARE KIT

FOLLOW THIS STEP BY STEP INSTRUCTION GUIDE¹

GETTING STARTED

Attach yankauer to suction handle



REMEMBER

- Use Yankauer as needed during kit use (24 hours)
- Suction Handle attaches to ALL components in Kit
- For a secure fit with suction handle, TWIST COMPONENTS 1/4 TURN
- DO NOT THROW AWAY SUCTION HANDLE or YANKAUER during kit use (24 hours)



STEP 1

Time: _____

Clinician: _____



OPEN PURPLE TOOTHBRUSH PACK

- CONNECT Toothbrush to Handle, DIP in solution cup
- BRUSH teeth 1-2 minutes with Q2 kit solution², and SUCTION often
- MOISTURIZE lips with Oral Swab and Mouth Moisturizer
- DISCARD Toothbrush and Swabs, and REATTACH Yankauer to Handle
- CONNECT the suction catheter to Handle
- SUCTION
- DISCARD suction catheter and REATTACH Yankauer to Handle

STEP 2

Time: _____

Clinician: _____



OPEN BLUE SUCTION SWAB PACK

- CONNECT Suction Swab to Handle, DIP in solution cup
- SWAB mouth, and SUCTION often
- MOISTURIZE lips with Oral Swab and Mouth Moisturizer
- DISCARD Swabs, and REATTACH Yankauer to Handle

STEP 3

Time: _____

Clinician: _____



OPEN GREEN SUCTION PACK

- CONNECT Suction Swab to Handle, DIP in solution cup
- SWAB mouth, and SUCTION often
- MOISTURIZE lips with Oral Swab and Mouth Moisturizer
- DISCARD Swabs, and REATTACH Yankauer to Handle

STEP 4

REPEAT STEP 2 WITH BLUE SUCTION PACK

Time: _____

Clinician: _____

STEP 5

REPEAT STEP 3 WITH GREEN SUCTION PACK

Time: _____

Clinician: _____

STEP 6

REPEAT STEP 2 WITH BLUE SUCTION PACK

Time: _____

Clinician: _____

STEP 7

REPEAT STEPS 1 THROUGH 6

1. See Order of Use Guide on back panel of Q2 Oral Care kit
2. CHG or CPC

There are inherent risks in all medical devices. Please refer to the product labeling for indications, cautions, warnings and contraindications. Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. Non contractual document. The manufacturer reserves the right, without prior notice, to modify the products in order to improve their quality.