EN: FOR USE WITH YSI 400 SERIES ELECTRONICS ONLY

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	MR UNSAFE
	MEDICAL DEVICE
	STERILIZED WITH ETHYLENE OXIDE
	SINGLE STERILE BARRIER SYSTEM
	DO NOT REUSE
	DO NOT RESTERILIZE
	DO NOT USE IF PACKAGE IS DAMAGED
	NOT MADE WITH NATURAL RUBBER LATEX

	RX ONLY
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

	IMPORTANT INFORMATION
	Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury.

	INTENDED USE
	The AirLife Esophageal Stethoscope with Temperature Sensor is to be used for routine monitoring of core body temperature, as well as to monitor heart and respiratory sounds in an anesthetized patient by inserting the stethoscope tube into the esophagus.

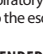
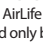
	INTENDED USERS
	The AirLife Esophageal Stethoscope with Temperature Sensor is intended to be used only by a licensed practitioner or trained healthcare professionals.

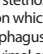
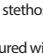
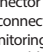
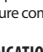
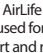
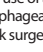
	PATIENT TARGET GROUP(S)
	The intended patient population is pediatric and adults. Patient must be anesthetized.

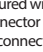
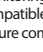
	PRODUCT DESCRIPTION
	The stethoscope portion of the device consists of a small diameter plastic tube upon which a flexible blue cuff is placed. When the device is inserted into the esophagus, the cuff facilitates transmission of sound to a male luer fitting at the proximal end of the tubing to which standard stethoscopes may be attached. The stethoscope is available in 9, 12, 18, and 24 French sizes.

	SECURED WITHIN THE LUMEN OF THE TUBING IS A TEMPERATURE MONITORING SENSOR. THE CONNECTOR AT THE PROXIMAL END INTERFACES WITH MANUFACTURER-BRANDED CABLES FOR CONNECTION TO A MONITORING UNIT TO DISPLAY TEMPERATURE. THE TEMPERATURE MONITORING SENSORS IS AVAILABLE IN 400 CONFIGURATIONS FOR CONNECTION TO A COMPATIBLE MONITOR. CONSULT THE USER MANUAL FOR THE PATIENT MONITOR TO ENSURE COMPATIBILITY.
--	--

	INDICATIONS
	The AirLife Esophageal Stethoscope with Temperature Sensor is indicated to be used for routine monitoring of core body temperature, as well as to monitor heart and respiratory sounds in an anesthetized patient by inserting the stethoscope tube into the esophagus.

	CONTRAINDICATIONS
	The use of an esophageal stethoscope may be contraindicated in patients with esophageal diverticulum or stenosis, in neonates or small infants undergoing neck surgery, in patients undergoing a tracheostomy or insertion of an internal jugular catheter, and in patients with conditions deemed unsafe for insertion of the stethoscope into the esophagus.

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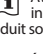
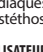
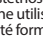
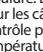
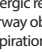
	WARNINGS
	Tracheal or bronchial insertion of stethoscope may cause airway obstruction and may be fatal if uncorrected. Correct placement may be verified with a laryngoscope.
	Carefully inspect the packaging and device for damage or defect. DO NOT use if the packaging is open or the device is damaged.
	This is a single use device. It is not intended for disinfection and/or subsequent re-use. Re-use could result in microbial contamination and potential patient injury.
	This device has not been evaluated for reprocessing or re-sterilization. Reprocessing and/or re-sterilization may damage the device, rendering it unusable and/or may lead to device failure.

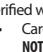
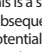
	ADVERSE REACTIONS
	While adverse reactions are rare, the following are potential adverse reactions associated with the use of temperature probes during insertion or while the device is in use:
	infection/sepsis
	allergic reaction to materials
	airway obstruction
	aspiration pneumonitis
	bronchial insertion
	electrical burns and shock
	epistaxis
	esophageal abrasion
	esophageal perforation
	trauma to pharynx

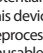
	DIRECTIONS FOR USE
	Remove the AirLife Esophageal Stethoscope with Temperature Sensor from its sterile package using aseptic technique.
	Verify the compatibility of the probe, interface cable, and patient monitor before use.
	Lubricate the cuff with a suitable water soluble lubricant.
	Insert the stethoscope and guide it into the esophagus.
	Listen for airway leaks when inserting. Invasion of the trachea may permit air to escape around the tracheal tube cuff. Do not seal the leak. Remove the esophageal stethoscope and reinsert. To verify proper placement, use a laryngoscope.
	Adjust the stethoscopes depth in the esophagus to maximize the heart and respiratory sounds.
	Connect the temperature probe to the appropriate reusable temperature cable from AirLife. Connect the cable to the patient monitor. Secure the cable using the strap clip.
	Follow the directions for use of the patient temperature monitor instrument.
	Remove the probe prior to extubation of the tracheal tube following accepted medical technique.
	Disconnect the probe (at the connector) and discard the probe. To disconnect, grasp both connectors firmly and pull. DO NOT pull cable or wire.

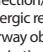
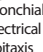
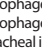
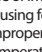
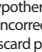
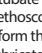
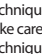
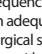
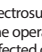
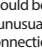
	SAFETY IN MRI NOT EVALUATED
	The AirLife Esophageal Stethoscope with Temperature Sensor's safety in MRI has not been evaluated, hence, the probe is considered MR Unsafe.

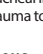
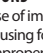
	OPERATING SPECIFICATIONS
	Operating Temperature: 25° C to 45° C
	Rated Output Range: 35° C to 42° C
	Rated Extended Output Range: 25° C to 35° C and 42° C to 45° C
	The reference body site is the core body temperature
	The measuring site is the esophagus
	Accuracy in Rated Output Range: +/- 0.2° C
	Accuracy in Rated Extended Output Range: +/- 0.3° C
	Time response
	The temperature probe has a direct mode of operation
	The maximum heating transient time is 2 minutes 45 seconds
	The maximum cooling transient time is 1 minute 45 seconds

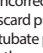
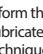
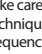
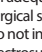
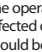
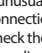
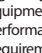
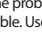
	DISPOSAL
	Dispose of any used device according to local, state, and federal laws and regulations. For safe disposal of devices, follow your facility's protocol.

	STORAGE AND TRANSPORT CONDITIONS
	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

	WARRANTY
	DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

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