EN AirLife™ Infant Skin Temperature Probe, Reusable

Instructions for Use

REF	<div> <div> </div> <div> <div></div> <div> </div> </div> </div>	Follow instructions for use	Non-sterile
LOT	<div> <div> </div> <div> <div></div> <div> </div> </div> </div>	Date of manufacture	Temperature limit
QTY	<div> <div> </div> <div> <div></div> <div> </div> </div> </div>	MR unsafe	Not made with natural rubber latex
MD	<div> <div> </div> <div> <div></div> <div> </div> </div> </div>	Medical device	UDI
U.S. Federal law restricts this device to sale by the order of a	Waste electrical & electronic equipment		

The following temperature probes are intended to be used in monitoring temperature. They are designed for use with GE Healthcare Graffite™ and Panda™ systems in neonatal intensive care unit. The reusable temperature products are intended to be reused after cleaning and disinfection. These devices are indicated for use by qualified medical personnel only.

Contraindications
The adhesive cover on this probe should never be applied to a skin surface that has been damaged by abrasion, laceration, incision or burning. The use of this skin probe cover on an infant, child or adult skin is contraindicated.

Instructions for use
Measuring site: Infant skin, preferably abdominal area

Ensure skin is clean and dry prior to application of probe.

To attach the reusable probe, use the heat reflecting patch. Place the metal cover on the probe and peel the paper backing from the adhesive side of the patch and place the patch over the probe with the reflective foil side up.

Plug the probe wire into the jack panel of the host device. Refer to the operator manual for the probe for detailed instructions on temperature measurement with the device.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myairlife.com and the competent authority of the Member State in which the user and/or patient is established.

Maintenance
Before use, check that the product is intact and clean. Manufacturer has successfully tested the enclosed reprocessing instructions for 20 cycles.

Warnings
-MR unsafe. Do not use this product in MRI environment.
- Before use, check that the product and its package is intact and clean. Do not use product if you see any visual signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, modify a product or use a product repaired by others.
- Do not clean this probe with any of the cleaning agents those listed in "Intended Use" may result in a hazardous condition and measurement errors.
- Use heat reflecting patch to avoid the effect of ambient temperature on measurement. Unusual temperature readings should be checked.
- Products are supplied non-sterile and must be unpacked, cleaned and sterilized prior to first use. Products must be reprocessed prior to each use.
- Adequate cleaning is a necessary requirement for subsequent effective sterilization of reusable equipment.

- Only use equipment and product-specific validated procedure for cleaning and sterilization.
- The equipment and accessories must be maintained and checked regularly according to the manufacturer’s instructions.
- Additionally all applicable laws, standards and guidelines for reprocessing must be followed.
- The use of personal protective equipment is the responsibility of the user.
- Deviation from the defined procedures within this instruction is the responsibility of the user.
- Manufacturer is not responsible for damages resulting from improper reprocessing. It rests with the user to validate their procedures and to comply with the validated parameters during every reprocessing.

- Do not strain the probe excessively during reprocessing.
- Do not pull on the probe cable when disconnecting the probe from host device.
- Do not reuse the disposable heat reflecting patch. Heat reflecting patch is disposable, reuse may cause a risk of cross-contamination or affect the measurement accuracy as a result of the product being physically damaged through excessive disinfection, or reuse.
- Do not remove the reflective foil from the reflective patch. Replace the patch when repositioning the probe or when adhesive strength degrades.
- Do not ensure appropriate safe functioning of the probe, check the availability of the console and with the host device and probe contact with skin both initially and during use. Use heat reflecting patch to ensure appropriate contact of probe with skin.
- Check appropriate placement and positioning of the probe and probe cable initially and during use to prevent disconnection of probe and probe connector, entanglement, pressure, tripping, etc. to occur as they might cause equipment malfunctions, airway obstruction, pain, irritation, discomfort, falling of other items or people, etc.

Pre-Processing Instructions
1. Ensure all pre-processing instructions are followed prior to cleaning and disinfecting.
2. Initial cleaning and disinfection of device as soon as possible after use (recommended within 1 hour).
3. Transport device using the institution’s established transport procedure.
4. Remove excess gross soil as soon as possible after use by wiping the probe.

5. Device shall be cleaned and disinfected utilizing the methods outlined below.

Cleaning Instructions
1. Ensure all pre-processing instructions are followed prior to cleaning.
2. Prepare the enzymatic cleaning solution by adding 20g water to achieve a temperature range of 27°C to 44°C (81°F to 111°F), per manufacturer’s instructions.
3. Use a soft clean cloth saturated with the cleaning solution to manually clean the surfaces of the devices including cracks and crevices for a minimum of 5 minutes. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.
4. Using a sterile, soft cloth saturated with purified water (PURW) at 27°C–44°C, manually rinse by wiping device for 30 seconds. Repeat two (2) additional times for a total of three (3) total rinse cycles.
5. Drying: Dry the device with a soft, sterile cloth.
6. Visually examine each device for cleanliness.
7. If necessary, repeat cleaning procedure until the device is thoroughly clean.

Disinfection
Intermediate level disinfection instructions
1. Prepare a 2% Glutaraldehyde disinfectant solution (CIDEX® or equivalent) according to the manufacturer’s directions.
2. Using a sterile, soft cloth saturated with the disinfection solution, manually disinfect all contaminated surfaces of the probe for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.
4. Using a sterile, soft cloth saturated with purified water (PURW) at 27°C–44°C, manually rinse by wiping device for 30 seconds. Repeat two (2) additional times for a total of three (3) rinse cycles, utilizing fresh purified water each time.
5. Dry the device using a sterile, lint-free cloth.

Technical information
Operating range: -30°C/-22°F to +60°C/+140°F
Accuracy: ±0.1°C (±0.30°C to ±0.2°C)

Storage
Storage Temperature -30°C/-22°F to +60°C/+140°F

Disposal
Reprocess products prior to disposal. Dispose electrical and electronic devices in compliance with WEEE or disposal local and state environmental regulations.

Technical description
For technical description see the host device user guide or manual, or contact your local sales representative.
Not made with natural rubber latex.

Ordering information
For further information, please contact your local sales representative or visit www.MyAirLife.com

REF	<div> <div> </div> <div> <div></div> <div> </div> </div> </div>	Follow instructions for use	Non-sterile
LOT	<div> <div> </div> <div> <div></div> <div> </div> </div> </div>	Date of manufacture	Temperature limit
QTY	<div> <div> </div> <div> <div></div> <div> </div> </div> </div>	MR unsafe	Not made with natural rubber latex
MD	<div> <div> </div> <div> <div></div> <div> </div> </div> </div>	Medical device	UDI
U.S. Federal law restricts this device to sale by the order of a	Waste electrical & electronic equipment		

AirLife Finland Oy
Kuortaneenkatu 2
FI-000510 Helsinki,
Finland
www.MyAirLife.com

Made in Mexico
Customer Service:
productquality@myairlife.com

Australian Sponsor
Airlife Australia Holdings Pty Ltd.
PO BOX 97
NORTH RYDE BC NSW 1670
Australia

0000000741 Rev A 202506

AirLife Finland Oy
Kuortaneenkatu 2
FI-000510 Helsinki,
Finland
www.MyAirLife.com

Made in Mexico
Customer Service:
productquality@myairlife.com

Australian Sponsor
Airlife Australia Holdings Pty Ltd.
PO BOX 97
NORTH RYDE BC NSW 1670
Australia

0000000741 Rev A 202506

AirLife Finland Oy
Kuortaneenkatu 2
FI-000510 Helsinki,
Finland
www.MyAirLife.com

Made in Mexico
Customer Service:
productquality@myairlife.com

Australian Sponsor
Airlife Australia Holdings Pty Ltd.
PO BOX 97
NORTH RYDE BC NSW 1670
Australia

0000000741 Rev A 202506

AirLife Finland Oy
Kuortaneenkatu 2
FI-000510 Helsinki,
Finland
www.MyAirLife.com

Made in Mexico
Customer Service:
productquality@myairlife.com

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