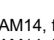
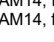
Vital Signs™ Diagnostic Cardiology Communication Host Cables Instructions for Use

REF			
2016560-001	Vital Signs™ Cable for CAM14, for MAC500™, 1.3 m/4.2 ft		Non-sterile
2016560-002	Vital Signs™ Cable for CAM14, for CASE™, 4.6 m/15 ft		Waste electrical & electronic equipment
2016560-003	Vital Signs™ Cable for CAM14, for MAC500™, 4.6 m/15 ft		Follow instructions for use
2016560-004	Vital Signs™ Cable for CAM14, for CASE™, 4.6 m/15 ft		Temperature limit
QTY	Quantity		MR unsafe
Manufacturer	Manufacturer		Not made with natural rubber latex.
Date of manufacture	U.S. Federal law restricts this device to sale by or on the order of a physician		UDI Unique device identifier
MD	Medical device		

Intended use
This diagnostic Cardiology Communication Host Cables provide data transfer between GE Healthcare Diagnostic Cardiology platforms and CAM 14 acquisition module. Use is limited by the indications for use of the connected diagnostic endpoint. This device is intended to be used by trained operators in a medical professional's environment or under the direction of a trained medical professional.

Instructions for use
Before use, check that the product is intact and clean. For compatibility and information regarding operating the device, see the device user manual. For use with GE Healthcare equipment.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myairlife.com and the competent authority of the Member State in which the user and/or patient is established.

Maintenance
Before use, check that the product is intact and clean. Manufacturer has successfully tested the enclosed reprocessing instructions for 80 cycles.

Warnings
Do not connect conductive connector parts contact any other conductive parts including earth.
Check appropriate placement or position of the cable initially and regularly to ensure it remains in place to prevent entanglement, twisting, tripping, etc. To occur as they might cause airway obstruction, irritation, decubitus, falling of other items or people, etc.

Cautions
Products are supplied non-sterile and must be unpacked, cleaned and disinfected prior to first use. Products must be reprocessed prior to each use. Adequate cleaning is a necessary requirement for subsequent effective disinfection.
Only use equipment and product-specific validated procedure for cleaning and disinfecting.
The used equipment and accessories must be maintained and checked regularly according to the manufacturer's instructions.
Additional standards and guidelines for reprocessing must be followed.
The use of personal protective equipment is in the responsibility of the user.
Deviations from the defined procedures within this instruction is in the responsibility of the user.
Manufacturers is not responsible for damages resulting from improper reprocessing. It rests with the user to validate their procedures and to comply with the validated parameters during every reprocessing.
Manufacturers is not responsible for damage to the equipment caused by inappropriate use of the product for physical cleaning before use. Do not use product if you see any visual signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product for other purposes.
To ensure appropriate and safe functioning of the cable, check stability of connection with host devices initially and occasionally over time.

Pre-Processing Instructions
1. Ensure all pre-processing instructions are followed prior to cleaning and disinfecting.
2. Initiate cleaning and disinfection of the device as soon as possible after use (recommending within 1 hour).
3. Transport device in the institution's established transport procedure.
4. Remove excess gross soil as soon as possible after use by wiping the device.
5. Clean the cable and disinfected utilizing a minimum of one (1) disinfection method and one (1) disinfection method outlined below.

Instructions for pre-disinfection or disinfection
1. Use the disinfectant solution in the manufacturer's established or validated procedure to clean the device and disinfect it.
2. Transporirajte uređaj prema postupku ustanove za prevoz i transport na instituciju.

Cleaning Instructions
1. Ensure all pre-processing instructions are followed prior to cleaning.
2. Prepare the enzymatic / neutral pH detergent, utilizing tap water entailing a temperature range of 27°C to 44°C (81°F to 111°F), per manufacturer's instructions.
3. Use a soft, clean cloth saturated with the cleaning solution to manually clean all surfaces of the device including cracks and crevices for a minimum of 30 seconds. In conjunction with this step, use an appropriately sized brush (soft-bristled), to remove all visible soil from the device paying particular attention to cracks, crevices and other hard to clean areas.
Note: It is recommended that enzymatic or neutral pH detergent solution should be changed when it becomes grossly contaminated (bloody and/or discolored).
4. Rinse the device thoroughly by using a soft, clean cloth saturated with lukewarm tap water with a temperate range of 27°C to 44°C (81°F to 111°F) for a minimum of 30 seconds. Repeat two (2) additional times for a total of three (3) total rinse cycles.
5. Drying: Dry with a sterile, soft cloth.
6. Visually inspect each device for cleanliness.
7. If visible soil remains, repeat cleaning procedure until the device is thoroughly clean.

Disinfection Intermediate level disinfection instructions (option 1)
1. Prepare a 2% disinfectant solution (CIDE® or equivalent) according to the manufacturer's directions.
2. Using a sterile, soft cloth saturated with the disinfection solution, manually clean all contaminated surfaces of the device for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfct hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.
4. Using a sterile, soft cloth saturated with purified water (PURW) at 27°C – 44°C, manually rinse by wiping the device for 30 seconds. Repeat two (2) additional times for a total of three (3) rinse cycles, utilizing fresh purified water each time.
5. Drying: Dry the device using a sterile, lint-free cloth.

Intermediate level disinfection instructions (option 2)
1. Prepare a 70% IPA disinfectant solution according to the manufacturer's directions.
2. Using a sterile, gauze saturated with the disinfection solution, manually clean all contaminated surfaces of the device for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfct hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.
4. Using a sterile, gauze saturated with purified water (PURW) at 27°C – 44°C, manually rinse by wiping the device for 30 seconds. Repeat two (2) additional times for a total of three (3) rinse cycles, utilizing fresh purified water each time.
5. Drying: Dry the device using a sterile, lint-free cloth.

High level disinfection instructions (option 3)
1. Prepare a 70% IPA disinfectant solution according to the manufacturer's directions.
2. Using a sterile, gauze saturated with the disinfection solution, manually clean all contaminated surfaces of the device for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfct hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.
4. Using a sterile, gauze saturated with purified water (PURW) at 27°C – 44°C, manually rinse by wiping the device for 30 seconds. Repeat two (2) additional times for a total of three (3) rinse cycles, utilizing fresh purified water each time.
5. Drying: Dry the device using a sterile, lint-free cloth.

Customer Service:
productquality@myairlife.com
Australasian Supplier:
AirLife Australia Holdings Pty Ltd.
PO BOX 97
NORTH RYDE BC NSW 1670
Australia

Finland Oy
Kuortaneentie 2
FI-00100 Helsinki, Finland
www.myairlife.com

Customer Service:
productquality@myairlife.com
Australasian Supplier:
AirLife Australia Holdings Pty Ltd.
PO BOX 97
NORTH RYDE BC NSW 1670
Australia

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