## Multi-Link™ DIN Converters

### Instructions for Use

|   |   |
|---|---|
| REF                                     | 4137-001 Multi-Link™ DIN Converter, 3-Lead 4137-001 Multi-Link™ DIN Converter, 5-Lead 4137-012 Multi-Link™ DIN Converter, 10-Lead |
| REF                                     | Catalogue number  |
| LOT                                     | Lot number  |
| MD                                      | Medical device  |
| QTY                                     | Quantity  |
| Manufacturer                            | Temperature limit   |
| Date of manufacture                     | Not made with natural rubber latex.   |
| MR unsafe                               | UDI Unique device identifier  |
| Waste electrical & electronic equipment | U.S. Federal law restricts this device to use only on the order of a physician.   |

### Intended use

Multi-Link™ Cable and Lead Wire Systems are reusable electrode cable systems designed to transmit signals from patient electrodes to various electrocardiograph recorders/monitors for both diagnostic and monitoring purposes.

### Instructions for use

Use these DIN converters to connect ECG DIN leads/in to Multi-Link ECG Cables.

These products are indicated for use by qualified medical personnel only.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myairlife.com and the competent authority of the member State in which the user's avtor patient is established.

### Maintenance

Before use, check that the product is intact and clean. Manufacturer has successfully tested the enclosed reprocessing instructions for 30 cycles.

### Warnings

- Do not use this product in MRI environment.
- Do not let the conductive product parts contact any other conductive parts including earth.
- Avoid direct contact of the electrocsgury unit neutral electrode to the patient to avoid burns at monitor measurement sites. Do not place ECG cables or leads/wires close to the electrocsgury unit neutral electrode or associated chest area.
- Do not allow humidity or liquid to enter the converter.
- Inspect the product for physical damage before use. Do not use the product or any accessories if physical damage is observed. Discard or replace the discarded immediately. Never try to repair a damaged product. Do not use the product if it is damaged.
- Adequate cleaning is a necessary requirement for subsequent effective use.
- Only use equipment and product-specific validated procedures for cleaning and disinfection.
- Only use equipment and accessories must be maintained and checked regularly according to the manufacturer's instructions.

- Additional applicable laws, standards and guidelines for reprocessing must be followed.
- The use of personal protective equipment is the responsibility of the user.
- Deviation from the defined procedures within this instruction is the responsibility of the user.
- The manufacturer is not responsible for damages resulting from improper reprocessing. It rests with the user to validate their procedures and to comply with the validated parameters during every reprocessing.
- Use only the product and liquid combinations as mentioned in this IFU for proper and accurate functioning of measurement. Using components or monitors other than the ones referenced might result in a measurement error or hazardous connection. Observe connector part color coding and/or markings on products to ensure appropriate compatibility with monitor and other accessories.
- Ensure appropriate and safe functioning of the product, check location of connection with monitor and/or other accessories initially and occasionally over time.
- Check appropriate placement or position of the product initially and occasionally over time to prevent entanglement, pressure, tipping, etc., to occur as they might cause avoid injury, irritation, dislodging, falling of other items or people.

### Pre-processing Instructions

- Ensure all pre-processing instructions are followed prior to cleaning and disinfecting.
- Initial cleaning and disinfection of device as soon as possible after use (recommended within 1 hour).
- Transport device using the institution's established transport procedure.
- Remove excess gross soil as soon as possible after use by wiping the device.
- Device shall be cleaned and disinfected utilizing a minimum of one (1) cleaning method and one (1) disinfection method outlined below.

### Cleaning Instructions

- Ensure all pre-processing instructions are followed prior to cleaning.
- Prepare the enzymatic, alkaline detergent cleaning spray water at a cleaning temperature range of 27°C to 44°C (81°F to 111°F), per manufacturer's instructions.
- Use a soft, clean cloth saturated with the cleaning solution to manually clean all surfaces of the devices including cracks and crevices for a minimum of 5 minutes. In conjunction with this step, use an appropriately sized brush (soft-bristled), to remove any visible soil from the device paying particular attention to cracks and crevices and other hard to clean areas.
- Note: It is recommended that enzymatic or neutral pH detergent solution should be changed when it becomes grossly contaminated (bloody and/or soiled) after a step.
- Rinse the device thoroughly by using a soft, clean cloth saturated with lukewarm tap water with a temperate range of 27°C to 44°C (81°F to 111°F) for a minimum of 30 seconds. Repeat step 2 (additional times) for a total of three (3) total rinse cycles.
- Drying: Dry the device with a soft, lint-free cloth.

**Disinfection**  
**Intermediate level disinfection instructions (option 1)**  
1. Prepare a 70% IPA disinfectant solution according to the manufacturer's directions.
2. Using a sterile, soft cloth saturated with the disinfection solution, manually clean all contaminated surfaces of the device for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

- Using a sterile, gauze saturated with purified water (PURW) at 27°C – 44°C, manually rinse by wiping the device for 30 seconds. Repeat step 2 (additional times) for a total of three (3) rinse cycles, utilizing fresh purified water each time.
  - Drying: Dry the device using a sterile, lint-free cloth.
- Disinfection**  
**Intermediate level disinfection instructions (option 2)**  
1. Prepare a 70% IPA disinfectant solution according to the manufacturer's directions.
2. Using a sterile, gauze saturated with the disinfection solution, manually disinfct all contaminated surfaces of the device for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

**Disinfection**  
**High level disinfection instructions (option 3)**  
1. Prepare a 2% Glutaraldehyde disinfection solution according to the manufacturer's directions.
2. Using a sterile, gauze saturated with the disinfection solution, manually disinfct all contaminated surfaces of the device for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

**Disinfection**  
**High level disinfection instructions (option 4)**  
1. Prepare a 2% Glutaraldehyde disinfection solution according to the manufacturer's directions.
2. Using a sterile, gauze saturated with the disinfection solution, manually disinfct all contaminated surfaces of the device for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

**Instructions for disinfection at maximum level (option 2)**  
1. Prepare a 2% Glutaraldehyde disinfection solution according to the manufacturer's directions.
2. Using a sterile, gauze saturated with the disinfection solution, manually disinfct all contaminated surfaces of the device for a minimum of 15 minutes.
3. In combination with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

**Instructions for disinfection at maximum level (option 3)**  
1. Prepare a 2% Glutaraldehyde disinfection solution according to the manufacturer's directions.
2. Using a sterile, gauze saturated with the disinfection solution, manually disinfct all contaminated surfaces of the device for a minimum of 15 minutes.
3. In combination with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

**Instructions for disinfection at maximum level (option 4)**  
1. Prepare a 2% Glutaraldehyde disinfection solution according to the manufacturer's directions.
2. Using a sterile, gauze saturated with the disinfection solution, manually disinfct all contaminated surfaces of the device for a minimum of 15 minutes.
3. In combination with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lumen brush) to disinfect hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

- Using a sterile, gauze saturated with purified water (PURW) at 27°C – 44°C, manually rinse by wiping the device for 30 seconds. Repeat step 2 (additional times) for a total of three (3) rinse cycles, utilizing fresh purified water each time.
- Drying: Dry the device using a sterile, lint-free cloth.

**Storage**  
Storage Temperature: -30°C/-22°F to +60°C/+140°F

**Disposal**  
Reprocess products prior to disposal. Dispose electrical and electronic devices in compliance with WEEE or dispose locally and in accordance with environmental regulations. This product does not contain PVC.

**Technical description**  
- The product meets the requirements of the standard, *ECG trunk cables and patient leads/wires (ANSI/AAMI EC53)*. This product is intended for multi-patient use.
- Not made with natural rubber latex.

**Ordering information**  
For more information, please contact your local sales representative or visit www.myAirLife.com.

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