[EN]

## Vital Signs™ Anesthesia gas sampling line, disposable

with female Luer connector


Instructions for Use

**[REF]** 73325-HEL Vital Signs™ gas sampling line, MF Luer, disposable, 3m/10ft

<b>[REF]</b> Catalogue number	 Consult instructions for use
<b>[LOT]</b> Lot number	 Follow instructions for use
<b>[QTY]</b> Quantity	 Single use
 Manufacturer	 U.S. Federal law restricts this device to sale by or on the order of a physician.
 Date of manufacture	

**Intended use**
This gas sampling line can be used for monitoring CO<sub>2</sub>, O<sub>2</sub>, N<sub>2</sub>O and anesthetic agents. Use with water traps with male Luer gas sampling connector.

Products are in indirect patient contact via gas passing through sampling site. These products are indicated for use by qualified medical personnel only.

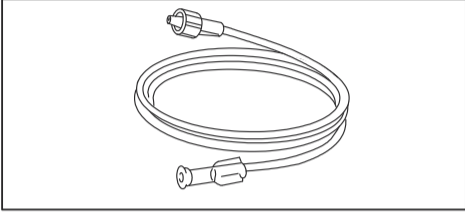**Material**
**PE** Polyethylene
**PVC** Polyvinyl chloride

**Instructions for use**

- Connect the sampling line to the monitor and the appropriate gas sampling port.
- Check all connections for leaks.

**Warning**

- No modification is allowed.
- Single-use products are not designed to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy and/or system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization and/or reuse.

**Disposal**
This product and package should be disposed of according to local environmental and waste disposal regulations.

**Technical description**

For technical description see the host device user guide or manual, or contact your local sales representative.

**Ordering information**

For further information, please contact your local sales representative or visit www.myAirLife.com.

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