

AIRFLOW™ RESUSCITATION BAGS

AF5140MB-PW

Size: Adult
 Label: Purple
 Body Mass: > 40 kg (88 lbs)
 Bag Volume: 1500 mL
 Stroke Volume: 700-780 mL
 Integrated manometer
 Pre-attached filter
 Pre-attached PEEP valve




AF2140MB-PW

Size: Pediatric
 Label: Orange
 Body Mass: ≤ 40 kg (88 lbs)
 Bag Volume: 1000 mL
 Stroke Volume: 590-600 mL
 Integrated manometer
 40 cm H₂O Pop-Off valve (use orange cap to override)
 Pre-attached filter
 Pre-attached PEEP valve



AF3140MC-PW

Size: Infant
 Label: Pink
 Body Mass: ≤ 10 kg (22 lbs)
 Bag Volume: 300 mL
 Stroke Volume: 150-170 mL
 Integrated manometer
 40 cm H₂O Pop-Off valve (use orange cap to override)
 Pre-attached filter
 Pre-attached PEEP valve

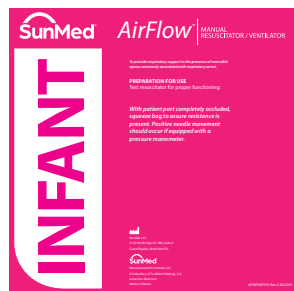


BEFORE USE:

1. Verify red tape on the bag string is not broken or damaged before opening.
2. The adult bags are packaged collapsed. After removing from the packaging, expand the bag to its full size (pinch at neck and tail and pull).



3. Remove rubber band from 7' kink resistant oxygen tubing and unfold oxygen reservoir to assure airflow is not restricted.

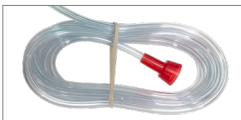

4. Occlude patient port and squeeze bag body to assure resistance and positive needle function on manometer.
5. Inspect mask for adequate inflation.
6. The pre-attached PEEP and filter can improve workflow efficiency and save time.
7. Connect red fits-all oxygen connector to oxygen and set flow to recommended flow rate
8. For supplemental oxygen, use an oxygen flow ≥ 15 LPM.
9. Follow Directions for Use