VIXONE™ INSTRUCTIONS FOR ASSEMBLY AND ADDING MEDICATION

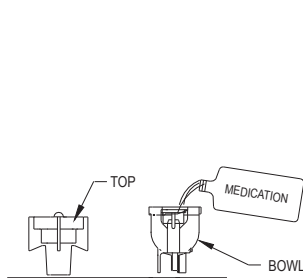
FIGURE 1

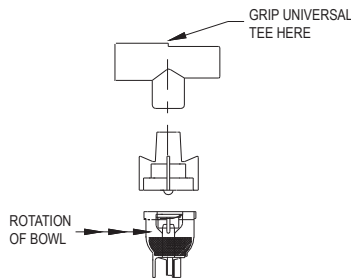
FIGURE 2

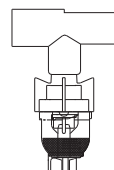
FIGURE 3

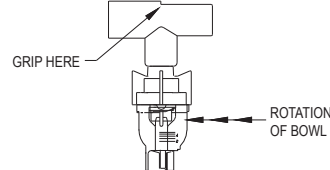
FIGURE 4

EN

1. The VixOne™ is packaged with the Top and Bowl unassembled to enhance its intended application. Medication may be added directly to the Nebulizer Bowl.
2. Tips for assembly of the Nebulizer components: Install the Universal Tee on the Top to provide a handle for easier assembly. Grip the top of the Tee and align the Top with the Bowl. Press the Nebulizer components together firmly*, ensuring that the Top and the Bowl contact surfaces are flush, and then rotate the Bowl until it locks in place. *NOTE: Some resistance to component assembly will be encountered, but this is essential in maintaining the desired seal integrity.
3. Configure Nebulizer for application by attaching prescribed auxiliary components. If a Mouthpiece is included in the kit, use the small polybag to wrap around the Mouthpiece during installation, to protect oral contact surfaces.
4. Unassemble the Nebulizer components for cleaning or addition of supplementary medication as follows: Grip the top of the Tee and rotate the Bowl slightly until the Top is released. Pull to separate components.

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