REF 13110, 13113

EN

English

Salter Labs™ Medium Concentration Oxygen Mask

DEVICE DESCRIPTION:

The device is non-sterile, disposable and for single-patient multi-use. An oxygen mask is a device placed over a patient's nose or mouth to administer oxygen.

INTENDED PURPOSE:

A Medium Concentration Oxygen Mask is a device placed over a patient's nose or mouth to administer oxygen.

INDICATIONS FOR USE:

For patients requiring supplemental oxygen. To treat hypoxia, relieve shortness of breath and pre-oxygenate for surgical procedures in spontaneously breathing patients.

ENVIRONMENT:

Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities.

INTENDED USER:

Trained medical professionals.

PATIENT TARGET GROUP:

Infant, pediatric, adult.

EXPECTED CLINICAL BENEFITS:

The oxygen mask delivers up to 15 LPM of oxygen to reverse or prevent hypoxemia, and provide relief of shortness of breath in oral and nasal breathers.

CONTRAINDICATIONS:

None Known.

⚠️ WARNINGS:

- If oxygen is in use, do not use near flame or heat source.
- Position tubing to avoid strangulation.
- Device contains components that may present choking hazard.
- Patient may become hypoxic if oxygen flow is interrupted.
- Use a flow rate ≥ 5 LPM to prevent CO₂ accumulation and re-breathing.
- Keep excess tubing loosely coiled and out of the way to prevent kinking and tripping hazard.
- To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device before connecting.

⚠️ CAUTIONS:

- Consult physician if patient develops infection, skin irritation or material sensitization.
- Do not place anything on supply tubing that may obstruct flow.
- This is a single-patient multi-use product, reuse on more than one patient may cause a risk of cross-contamination.

RESIDUAL RISKS:

Refer to Warnings and Cautions.

NECESSARY USER QUALIFICATIONS:

The end-user should receive training on the proper use of the device and be familiar with the indications for use, warnings, and cautions as stated in the IFU.

RX Only: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner.

INSTRUCTIONS FOR USE:

Follow instructions to avoid risk of infection and contamination.

- 1) Connect tubing to oxygen source.
- 2) Place mask over nose and mouth, adjust elastic strap and nose bar for a snug fit.
- 3) Check for leaks at connections and for proper placement prior to use.
- 4) Add humidification if nasal dryness or irritation occurs.

This product is disposable and is not intended for prolonged use greater than 14 days.

CLEANING/REUSE INSTRUCTIONS

A discontinued use of this product on the same patient, with periods of use separated by a period of non-use, is acceptable. The user must ensure that the product is not damaged or contaminated between usages on the same patient.

SAFE DISPOSAL

Decontaminate and dispose of all potentially biohazardous material. Dispose of device in accordance with local, state or national regulations.

INCIDENT REPORTING

Notice to the user that any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myairlife.com, and the competent authority of the Member State in which the user and/or patient is established. If the patient's physical symptoms worsen or experience a sudden change in their condition (e.g., increased shortness of breath, fever, dizziness) call the physician.

PERFORMANCE CHARACTERISTICS

- For use with Flow Rate 3 - 15 LPM
- For use with Bonded O₂ Tubing adaptor

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MT Promedt Consulting Ltd.
MedEnvoy UK Limited