Airway Adapters - Instructions for Use

Refer to the Instructions for Use accompanying your host monitor.

Note: Sterilization will likely compromise system performance.

Symbols

These symbols may be found on airway adapter labels, packaging, and other documentation.

	Pediatric/Adult only		This device complies with Regulation (EU) 2017/745		Storage humidity
	Neonatal/Pediatric only		U.S. Federal law restricts this device to sale by or on the order of a physician.		Storage temperature
	Consult Instructions for Use		Electronic instructions for use		Do not reuse
	Quantity		Medical device		MR Unsafe
	Importer		Authorised representative for Switzerland		Distributor

Part number	Description	Intended Use/Indications for Use	Instructions for Use
1149386	Pediatric/Adult Airway Adapter, Single use	For monitoring Pediatric/Adult patients with endotracheal tube size greater than 4.0mm with the CAPNOSTAT CO ₂ sensor.	1. Verify the adapter is intact. Press the CAPNOSTAT CO ₂ sensor onto the airway adapter. It will "click" into place when properly seated. 2. Perform an Airway Zero only if prompted by the monitor (see User's Manual for details).
1149448	Neonatal/Pediatric Airway Adapter, Single use	For monitoring Neonatal/Pediatric patients with endotracheal tube size of 4.0mm or smaller with the CAPNOSTAT CO ₂ sensor.	3. Install at the proximal end of the circuit between the elbow and the ventilator wye. If using an HME, place adapter between HME and wye. The cord of the CAPNOSTAT CO ₂ sensor should be facing away from the patient. For optimal results, DO NOT place the adapter between the ET tube and the elbow. 4. Ensure the integrity of the patient breathing circuit after insertion of the airway adapter.

Disposal

Dispose of in accordance with all applicable federal, state, and local medical / hazardous waste practices.

NOTE: Any serious incident that has occurred in relation to this device should be reported to Respiromics Novamatrix and the competent authority of the country in which the user and/or patient is established.

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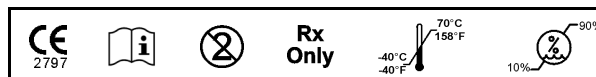
Australian Sponsor
AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW 1670
Australia

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